

LOCATOR®
You Just Can't Miss




- **Self-Aligning Design**
- **Attachment Durability**
- **Angle Correction Up To 40°**
- **Lowest Vertical Height**
- **Easy Maintenance**


You And Your Patients Just Can't Miss With LOCATOR®

LOCATOR OverdentSURE Abutment

The LOCATOR Abutment is ideal for overdenture restorations. Clinical use has shown that the self-locating design enables patients to align and seat their overdentures with ease. Independent testing* has shown that the LOCATOR Attachment resists wear and maintains satisfactory retention for as many as 56,000 cycles of function. The LOCATOR Abutment answers problems found with other abutments, such as limited interocclusal space, lack of attachment durability and difficulty of patient insertion. At 3.17mm total height, LOCATOR is 1.68mm lower than other available abutments.

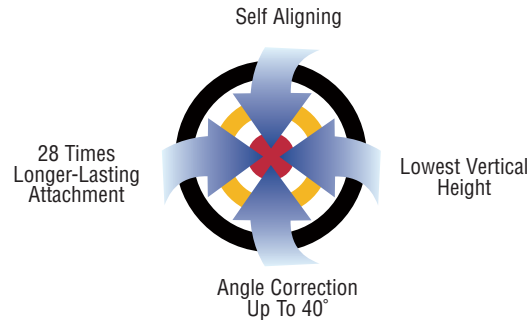

Self-Aligning Design

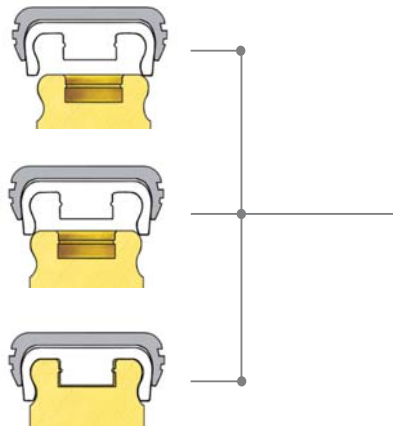


- **Makes It Easier For Patients To Position And Correctly Seat Their Overdentures Into Place.**
- **Reduces Problems Due To Insertion And Removal Of Overdentures In As Little As One Office Visit.**
- **Reduces Additional Wear And The Destruction Of Attachments From Improper Seating.**

Attachment Durability

- **Patented Dual-Retention Design Provides Approximately Twice The Amount Of Retention Surface Area Than Only Internal Retention Attachments.**
- **Combination Of Inside And Outside Retention Ensures Long-Lasting Retention Life.**
- **Independent Laboratory Testing* Demonstrated 56,000 Cycles Of Function With Excellent Retention.**
- **Independent Laboratory Testing* Showed The LOCATOR Male Attachment Is 28 Times Longer-Lasting, Compared To The ERA® Implant Attachment.**

*Delson Testing Laboratories was contracted to measure the retention of the LOCATOR Male Attachment, compared with the ERA® Implant Attachment. Testing done on nonsterile attachments. ERA is a registered trademark of Sterngold Attachments.

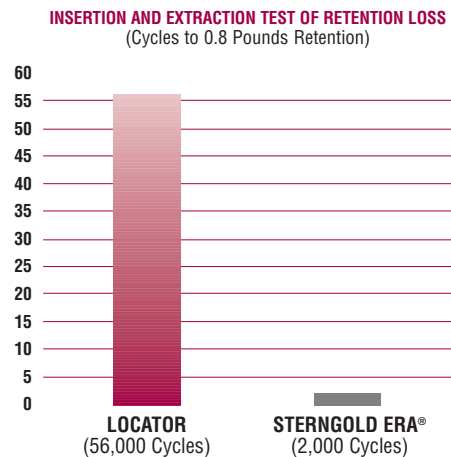

Angle Correction Up To 40°




- Allows Correction Of Angles Up To 40° Between Divergent Implants Using The Same Abutment.
- Eliminates Positioning Of Angled Abutments.
- Extended-Range Male Attachment Demonstrates The Same Long-Lasting Retention.
- Maintains A Low Vertical Height Of 3.17mm From Platform To Top Of Housing.

Lowest Vertical Height

- A Critical Advantage Where Interocclusal Space Is Limited.
- Total Abutment And Attachment Height Is Only 3.17mm With 1mm Abutments.
- A Full 1.68mm Lower Than The Next Lowest Competitive Abutment.




3.17mm LOCATOR®	4.85mm ERA®	5.82mm DaL-Ro	6.4mm O-Ring	6.22mm EDS
--------------------	----------------	------------------	-----------------	---------------

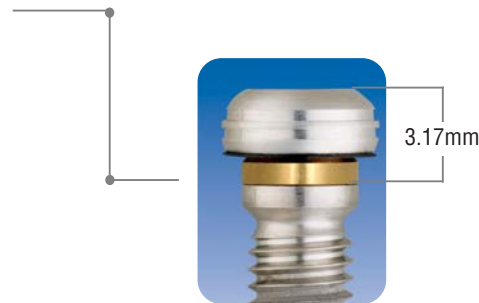

Easy Maintenance

New Design

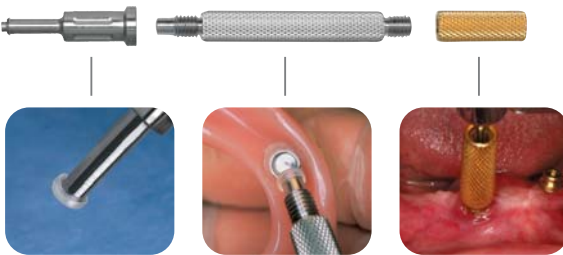

Attachment Removal

Attachment Insertion

Abutment Placement







- Attachment Durability Means Fewer Maintenance Office Visits.
- 3-In-1 Instrument Accomplishes All Aspects Of Attachment Replacement.
- Abutment Driver Is Compatible With ISO-Latch Torque Instruments Using A .050 Inch (1.25mm) Hex Driver Tip.
- Abutments Torqued To 20Ncm Means Reduced Loosening Complications.
- Triangle Driver Tip Also Available.

Patient Friendly And Practice Friendly

The LOCATOR Abutment's ease of use means increased patient satisfaction and it's features and durability mean decreased maintenance visits, so that you can devote greater time to increasing the profitability of your practice. If your target is lower maintenance on overdentures with higher patient satisfaction, you just can't miss with LOCATOR!

LOCATOR®








Better For Your Patients. Better For Your Practice®

LOCATOR Abutments		Collar Height	 Internal Connection	 External Connection
3.4mm Seating Surface  	1mm		IMLOA001	MLOA001
	2mm		IMLOA002	MLOA002
	3mm		IMLOA003	MLOA003
	4mm		IMLOA004	MLOA004
	5mm		IMLOA005	MLOA005
	6mm		IMLOA006	MLOA006
4.1mm Seating Surface  	1mm		ILOA001	LOA001
	2mm		ILOA002	LOA002
	3mm		ILOA003	LOA003
	4mm		ILOA004	LOA004
	5mm		ILOA005	LOA005
	6mm		ILOA006	LOA006


Included With Each Abutment:



LOCATOR Components

Impression Coping	Retention Males		
 LAIC1	 Replacement Males LARMS	 Light Retention Males LLRMS	 Extended Range Males LAERM
 LALA1	 Extra Light Retention Males LELRMS	 Extra Light Extended Range Males LELARM	

LOCATOR Instruments

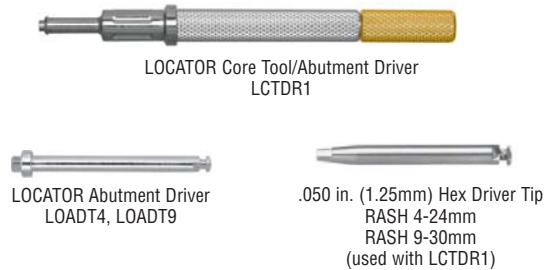

Global Headquarters
4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454
Outside The U.S.: +1-561-776-6700
Fax: +1-561-776-1272
www.biomet3i.com

Sign Up For BIOMET 3i's Electronic Newsletter "3innovations."
Simply Go Online To www.3i-online.com/signup

Better For Your Patients. Better For Your Practice. Certain and OSSEOTITE are registered trademarks of Implant Innovations, Inc. BIOMET 3i and design are trademarks and BIOMET is a registered trademark of BIOMET, Inc. LOCATOR is a registered trademark of Zest Anchors, Inc. ERA is a registered trademark of Sterngold Attachments. ©2007 BIOMET 3i, Inc. All rights reserved.



ART815
REV D 03/07

SUBSIDIARIES

AUSTRALIA
Phone: +61-2-9855-4444
Fax: +61-2-9888-9900

BELGIUM
Phone: +32-2-5410290
Fax: +32-2-5410291

BRAZIL
Phone: +55-11-5081-4405
Fax: +55-11-5081-7484

CANADA
Phone: +514-956-9843
Fax: +514-956-9844

FRANCE
Phone: +33-1-41054343
Fax: +33-1-41054340

GERMANY
Phone: +49-721-255177-10
Fax: +49-721-255177-73

IRELAND
Phone: +35-31-477-3925
Fax: +35-31-402-9590

MEXICO
Phone: +52-55-5679-1619
Fax: +52-55-5684-8098

THE NETHERLANDS
Phone: +31-(0)78-629-2800
Fax: +31-(0)78-629-2801

NEW ZEALAND
Phone: +64-508-122-221
Fax: +64-508-133-331

NORDIC REGION
Phone: +46-40-17-6090
Fax: +46-40-17-6099

PORTUGAL
Phone: +351-21-000-1645
Fax: +351-21-000-1675

SPAIN
Phone: +34-93-470-59-50
Fax: +34-93-372-11-25

SWITZERLAND
Phone: +41-1-3804646
Fax: +41-1-3834655

U.K.
Phone: +44-1628-829314
Fax: +44-1628-820182

DISTRIBUTORS

ARGENTINA
Dentalmax, SA
Phone: +54-1-482-71001
Fax: +54-1-482-67373

AUSTRIA
Wieladent
Phone: +43-7672-93901
Fax: +43-7672-93903

CHILE
Cybel, SA
Phone: +56-2-2321883
Fax: +56-2-2330176

CHINA
Atek Inc.
Phone: +86-21-6329-1265
Fax: +86-21-6329-1620

COLOMBIA
3i Colombia
Phone: +571-612-9362
Fax: +571-620-6412

COSTA RICA
Implantec S.A.
Phone: +506-2-256411
Fax: +506-2-247620

EL SALVADOR
Dentimer SA de CV
Phone: +503-263-6350
Fax: +503-263-6676

GREECE
Kostas Kornisorlis and Co.
Phone: +30-2310-269-079
Fax: +30-2310-555-573

ISRAEL
H.A. Systems
Phone: +972-3-6138777
Fax: +972-3-6138778

ITALY
Biomax, srl.
Phone: +39-0444-913410
Fax: +39-0444-913695

JAPAN
Implant Innovations Japan
Phone: +81-66-868-3012
Fax: +81-66-868-2444

KOREA
Jungsan Biomed Corp.
Phone: +82-2-516-1808
Fax: +82-2-514-9434

LEBANON
3i MENA s.a.l.
Middle East And North Africa
Phone: +961-1-694000
Fax: +961-1-694222

PARAGUAY
Andres H. Arce y Cia SRL
Phone: +595-21-208185
Fax: +595-21-496291

POLAND
Dental Depot
Phone: +48-71-341-3091
Fax: +48-71-343-6560

RUSSIA
Com-Dental
Phone: +7-495-797-6686
Fax: +7-499-242-9567

SINGAPORE
Asia Implant Support & Services
Phone: +65-6223-2229
Fax: +65-6220-3538

TAIWAN
Kuo Hwa Dental Suppliers Co., Ltd.
Phone: +886-2-2226-1770
Fax: +886-2-2226-8747

THAILAND
3i (Thailand) Co., LTD.
Phone: +662-252-6685
Fax: +662-252-6686

UKRAINE
Com-Dental
Phone: +38-067-70077667
Fax: +38-044-5017117

URUGUAY
ProImplant S.R.L.
Phone: +598-2-4034163
Fax: +598-2-4034163