



Rapid Adjustment. Enduring Strength. Aesthetic Design.
PreFormance® Provisional Components

- ***Easily Adjusted For Quick Preparation***
- ***Polymer Material With Titanium Interface For Strength***
- ***More Options For Clinicians – Cement And Screw-Retained Restorations – Pre-Angled Options***
- ***Use In DIEM® Immediate Non-Occlusal Loading Cases For Same Day Aesthetics***
- ***Available For Certain® And External Connection Implant Systems***



Provisionals For “PEEK” Performance

Today, your patients demand faster and more aesthetic results for their dental needs. That's why BIOMET **3i** developed the new **PreFormance® Provisional Post and Temporary Cylinder** for faster, more aesthetic, provisional restorations.

Rapid Adjustment, Enduring Strength And Aesthetic Design

The PreFormance Post and Temporary Cylinder are made of a strong, biocompatible material called PEEK (Polyetheretherketone) that is easily customized to your patient's tissue contours and anatomic profile. Intraoral preparation time is much faster and easier than with titanium and your patients can leave with **pleasing aesthetics the day of surgery.**

As a cost-effective option for optimum tissue healing, PreFormance Provisional Components offer some of the longest temporary in-vivo use of polymers on the market – 180 days versus 30 days.

As part of BIOMET **3i**'s DIEM® Procedure for complete dental care in one day, the PreFormance Provisional Post and Temporary Cylinder offer immediate aesthetics – another way for surgeons and clinicians to increase confidence, satisfaction and well-being for patients.



PreFormance Post Cement-Retained Provisional Option

- Available In Internal And External Connection For More Restorative Options
- The Certain® QuickSeat® Internal Connection Tells You The Abutment Is Properly Seated With An Audible And Tactile Click
- Available In Straight Or 15° Pre-Angled For Ideal Angle Correction
- Collar Height In 4 Or 6mm For More Natural Contours Without Trapping Cement
- Emergence Profile (EP®) For More Natural Aesthetics
- Smooth Surface For Easy Provisional Restoration Reline
- Flat Side For Anti-Rotation Of Restoration



PreFormance Temporary Cylinder Screw-Retained Provisional Option

- Available In Internal And External Connection For More Restorative Options
- The Certain QuickSeat Internal Connection Tells You The Abutment Is Properly Seated With An Audible And Tactile Click
- Knurled Surface For Mechanical Retention Of Resin Material
- Available In Hexed Or Non-Hexed For Single-Or Multiple-Unit Restorations



Rapid Adjustment To A Winning Smile

Cement-Retained Option

Follow these simple steps to cross the finish line to successful provisional restorations.



1. Select the appropriate PreFormance® Post. Thread the titanium screw through the post into the implant until finger tight. Mark the PreFormance Post with a pencil in areas requiring preparation.



2. Prepare margins to the selected depth. Follow the preferred gingival contours. The post can be prepared intraorally or on a laboratory abutment holder.



3. Once abutment preparation is complete, secure the post to the implant using the Titanium Hexed Screw torqued to 20Ncm using the Large Hex Driver Tip and a torque wrench.



4. Place a protective material over the screw head. Seal the access hole with temporary filling material. Fabricate a provisional crown on the PreFormance Post and cement using temporary cement.



Ordering Information



PreFormance® Post

Certain® PreFormance Abutments

Platform	Straight, 0° 4.0mm Collar	Straight, 0° 6.0mm Collar	PreAngled 15° 4.0mm Collar	PreAngled 15° 6.0mm Collar
----------	------------------------------	------------------------------	-------------------------------	-------------------------------

3.4mm				
3.8mm	IMPFP34	IMPFP36	IMPAPF34	IMPAPF36
4.1mm				
5.0mm	IPFP454	IPFP456	IPAPF454	IPAPF456
5.0mm				
5.0mm	IWFPF554	IWFPF556	IPAPF554	IPAPF556
6.0mm				
6.0mm	IWFPF664	IWFPF666	IPAPF664	IPAPF666



PreFormance Temporary Cylinder

Certain Temporary Cylinders

Platform	Hexed ●	Non-Hexed ○
3.4mm	IMPFTC34	IMPFTC32
4.1mm	IPFTC41	IPFTC42
5.0mm	IWFTC51	IWFTC52
6.0mm	IWFTC61	IWFTC62




PreFormance Post

External Connection PreFormance Abutments

Platform	Straight, 0° 4.0mm Collar	Straight, 0° 6.0mm Collar	PreAngled 15° 4.0mm Collar	PreAngled 15° 6.0mm Collar
----------	------------------------------	------------------------------	-------------------------------	-------------------------------

3.4mm				
3.8mm	MPFP34	MPFP36	MPAPF34	MPAPF36
4.1mm				
5.0mm	PFP454	PFP456	PAPF454	PAPF456
5.0mm				
5.0mm	WPFP554	WPFP556	PAPF554	PAPF556
6.0mm				
6.0mm	WPFP664	WPFP666	PAPF664	PAPF666

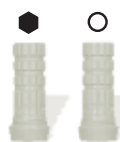

PreFormance Temporary Cylinder

External Connection Temporary Cylinders

Platform	Hexed ●	Non-Hexed ○
3.4mm	MPFTC34	MPFTC32
4.1mm	PFTC41	PFTC42
5.0mm	WPFTC51	WPFTC52
6.0mm	WPFTC61	WPFTC62



**For High Performance With Immediate Results,
Call Your BIOMET 3i Sales Representative Today**



Global Headquarters
4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454
Outside The U.S.: +1-561-776-6700
Fax: +1-561-776-1272
www.biomet3i.com

**Sign Up For BIOMET 3i's Electronic Newsletter "BIOMET 3innovations."
Simply Go Online To www.biomet3i.com/signup**



ART953
REV C 02/08

Certain, DIEM, EP, PreFormance and QuickSeat are registered trademarks of BIOMET 3i, Inc. BIOMET is a registered trademark and BIOMET 3i and design are trademarks of BIOMET, Inc. ©2008 BIOMET 3i, Inc. All rights reserved.

SUBSIDIARIES

AUSTRALIA

Phone: +61-2-9855-4444
Fax: +61-2-9888-9900

AUSTRIA

Phone: +43-(0)6235-200-45
Fax: +43-(0)6235-200-45-9

BELGIUM

Phone: +32-2-5410290
Fax: +32-2-5410291

BRAZIL

Phone: +55-11-5081-4405
Fax: +55-11-5081-7484

CANADA

Phone: +1-514-956-9843
Fax: +1-514-956-9844

FRANCE

Phone: +33-1-41054343
Fax: +33-1-41054340

GERMANY

Phone: +49-721-255177-10
Fax: +49-721-255177-73

IRELAND

Phone: +35-31-477-3925
Fax: +35-31-402-9590

MEXICO

Phone: +52-55-5679-1619
Fax: +52-55-5684-8098

THE NETHERLANDS

Phone: +31-(0)78-629-2800
Fax: +31-(0)78-629-2801

NEW ZEALAND

Phone: +64-508-122-221
Fax: +64-508-133-331

NORDIC REGION

Phone: +46-40-17-6090
Fax: +46-40-17-6099

PORTUGAL

Phone: +351-21-000-1645
Fax: +351-21-000-1675

SPAIN

Phone: +34-93-470-59-50
Fax: +34-93-372-11-25

SWITZERLAND

Phone: +41-44-380-46-46
Fax: +41-44-383-46 55

U.K.

Phone: +44-1628-829314
Fax: +44-1628-820182

DISTRIBUTORS

ARGENTINA

Dentalmax, SA
Phone: +541-1482-71001
Fax: +541-1482-67373

CHILE

Cybel, SA
Phone: +56-2-2321883
Fax: +56-2-2330176

CHINA

Atek Inc.
Phone: +86-21-6329-1265
Fax: +86-21-6329-1620

COLOMBIA

3i Colombia
Phone: +571-612-9362
Fax: +571-620-6412

COSTA RICA

Implantec S.A.
Phone: +506-2-256411
Fax: +506-2-247620

EL SALVADOR

Dentimer SA de CV
Phone: +503-263-6350
Fax: +503-263-6676

GREECE

Impladent Dental Implants, LLC
Phone: +30-2310-501-651
Fax: +30-2310-522-417

HONG KONG

Ositek Inc., Ltd.
Phone: +852-8121-6601
Fax: +852-3747-3754

ISRAEL

H.A. Systems
Phone: +972-3-6138777
Fax: +972-3-6138778

ITALY

Biomax, srl.
Phone: +39-0444-913410
Fax: +39-0444-913695

JAPAN

Implant Innovations Japan
Phone: +81-66-868-3012
Fax: +81-66-868-2444

KOREA

Jungsan Biomed Corp.
Phone: +82-2-514-9434
Fax: +82-2-514-9434

LEBANON

3i MENA s.a.l.
Middle East And North Africa
Phone: +961-1-694000
Fax: +961-1-694222

PARAGUAY

Andres H. Arce y Cia SRL
Phone: +595-21-208185
Fax: +595-21-496291

POLAND

Dental Depot
Phone: +48-71-341-3091
Fax: +48-71-343-6560

RUSSIA

Com-Dental
Phone: +7-495-797-6686
Fax: +7-499-242-9567

SINGAPORE

Asia Implant Support & Services
Phone: +65-6223-2229
Fax: +65-6220-3538

SOUTH AFRICA

Selective Surgical CC
Phone: +27-11-991-7007
Fax: +27-11-672-1391

TAIWAN

Kuo Hwa Dental Suppliers Co., Ltd.
Phone: +886-2-2226-1770
Fax: +886-2-2226-8747

THAILAND

3i (Thailand) Co., Ltd.
Phone: +662-252-6685
Fax: +662-252-6686

UKRAINE

Com-Dental
Phone: +38-067-7007667
Fax: +38-044-5017117

URUGUAY

Pro3implant S.R.L.
Phone: +598-2-4034163
Fax: +598-2-4034163