BOKET 31

The Final Piece Of The Aesthetic Puzzle

The Encode[®] Complete Restorative System

- Custom Milled CAD/CAM Abutments Are Anatomically Designed To Each Patient Scenario
- Robocast Technology Eliminates Implant-Level Impressions And Associated Component Inventory
- Working Above The Gingiva Helps To Preserve Soft Tissue



*Not Yet Commercially Available For Sale

The Puzzle Is Now Complete

Meeting aesthetic demands and increasing productivity can be challenging for clinicians. BIOMET *3i* now offers a new solution to achieve both. The BIOMET *3i* Encode[®] Complete Restorative System delivers a patient specific abutment with the margin height and natural emergence contours appropriate to meet the needs of the individual case. The procedure has been simplified due to the technology developed at BIOMET *3i*.

No Impression Copings Needed

With Encode Complete, all that is needed for you to receive a patient specific abutment and final restoration is:

- An Impression Of The Encode Healing Abutment And The Opposing Arch
- A Bite Registration
- Shade Selection

Codes embedded on the occlusal surface of the Encode Healing Abutment communicate the implant depth, hex-orientation, platform diameter and interface (Certain[®] Internal Connection or External Connection).



Maintenance Of Soft Tissue

Because there is no need to remove a healing abutment and seat an impression coping for an implant level impression, soft tissue is spared from the unnecessary trauma of traditional techniques. As a result, soft tissue levels may be preserved and bone remodeling potentially minimized.

Custom Designed With Precision

The result is an anatomically designed final Encode Abutment. Margin, contour, taper and emergence profile are custom crafted by BIOMET **3i** Master Dental Technicians to your specifications with consideration to the surrounding gingival architecture and dental structures.



Titanium Nitride

Zirconium*

*Not Yet Commercially Available For Sale


With Encode Complete, the clinician, the laboratory and BIOMET 3i come together to provide the patient with a custom aesthetic restoration.

A Custom Fit


The following Clinical Case Presentation demonstrates a 54-year-old female patient who presented with a failed fixed partial denture supported by teeth Nos. 11 and 13.




Tooth No. 13 (maxillary left second bicuspid) was fractured and deemed non-restorable.

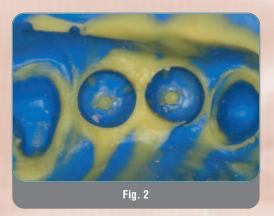


An impression was made of the Encode[®] Healing Abutments and natural tooth preparations.

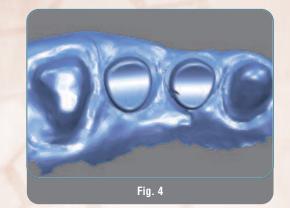

After scanning the casts, the abutments were designed virtually.

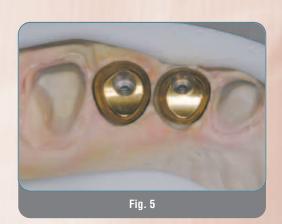

Implant analogs were placed into the cast using Robocast Technology allowing the definitive Encode Abutments to be placed on the master cast for fabrication of the definitive restorations by the laboratory.

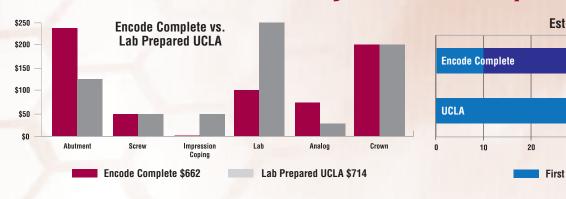

Time And Cost Analysis: Encode Complete

Roxel 31



The impression of the Encode® Healing Abutments was poured using a die stone for fabrication of a master cast. The casts were mounted on the recommended articulator using the bite registration. The work order was completed and the casts were sent to BIOMET **3** for fabrication of definitive Encode Abutments.






The titanium-nitride coated definitive Encode Abutments were seated into the internal interfaces of the implants with audible and tactile clicks, which ensured completed seating. The abutments were secured with Gold-Tite[®] Abutment Screws.



The definitive restorations were cemented and the patient was dismissed with oral hygiene instructions.

Photographs Courtesy Of: Dr. Christopher D. Ramsey Jupiter, FL

Restorative System vs. UCLA Abutment

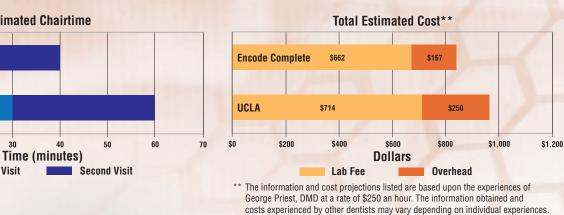

BOXET 31




The newest BIOMET *3i* Innovation includes robotic placement of implant analogs into the master cast in a process called Robocast Technology.



Robocast Technology eliminates the need for an implant level impression, which simplifies the procedure for more efficient use of valuable chairtime.

ARCHITECH PSR® Dental Technicians design the patient specific final abutment with appropriate margin height and natural emergence contours, according to the laboratory and clinician specifications. During the milling process, the virtual design data is sent to the Robocast Center for analog placement in the original Encode® Cast.



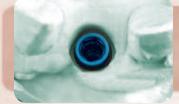

The result is an implant level master cast called Robocast, which the dental laboratory uses to fabricate the final restoration.

Encode[®] Healing Abutment Ordering Information

Ask Your Surgeon To Place Encode Healing Abutments

To Find Your Final Piece To The Aesthetic Puzzle, Call Your BIOMET 3i **Representative Today!**

3.4mm Seating Surface						
Emergence	Collar	Certain®	External Hex			
Profile	Height	Item Number	Item Number			
3.8mm	3.0mm	IEHA343	EHA343			
3.8mm	4.0mm	IEHA344	EHA344			
3.8mm	6.0mm	IEHA346	EHA346			
3.8mm	8.0mm	IEHA348	EHA348			
5.0mm	3.0mm	IEHA353	EHA353			
5.0mm	4.0mm	IEHA354	EHA354			
5.0mm	6.0mm	IEHA356	EHA356			
5.0mm	8.0mm	IEHA358	EHA358			
4.1mm Seating Surface						
Emergence	Collar	Certain	External Hex			
Profile	Height	Item Number	Item Number			
4.1mm	3.0mm	IEHA443	EHA443			
4.1mm	4.0mm	IEHA444	EHA444			
4.1mm	6.0mm	IEHA446	EHA446			
4.1mm	8.0mm	IEHA448	EHA448			
5.0mm	3.0mm	IEHA453	EHA453			
5.0mm	4.0mm	IEHA454	EHA454			
5.0mm	6.0mm	IEHA456	EHA456			
5.0mm	8.0mm	IEHA458	EHA458			
6.0mm	3.0mm	IEHA463	EHA463			
6.0mm	4.0mm	IEHA464	EHA464			
6.0mm	6.0mm	IEHA466	EHA466			
6.0mm	8.0mm	IEHA468	EHA468			
7.5mm	3.0mm	IEHA473	EHA473			
7.5mm	4.0mm	IEHA474	EHA474			
7.5mm	6.0mm	IEHA476	EHA476			
7.5mm	8.0mm	IEHA478	EHA478			

	5.0mm Seating Surface					
	Emergence	Collar	Certain	External Hex		
	Profile	Height	Item Number	Item Number		
	5.0mm	3.0mm	IEHA553	EHA553		
	5.0mm	4.0mm	IEHA554	EHA554		
	5.0mm	6.0mm	IEHA556	EHA556		
	5.0mm	8.0mm	IEHA558	EHA558		
	6.0mm	3.0mm	IEHA563	EHA563		
	6.0mm	4.0mm	IEHA564	EHA564		
	6.0mm	6.0mm	IEHA566	EHA566		
	6.0mm	8.0mm	IEHA568	EHA568		
	7.5mm	3.0mm	IEHA573	EHA573		
	7.5mm	4.0mm	IEHA574	EHA574		
	7.5mm	6.0mm	IEHA576	EHA576		
	7.5mm	8.0mm	IEHA578	EHA578		
6.0mm Seating Surface						
	Emergence	Collar	Certain	External Hex		
	Profile	Height	Item Number	Item Number		
	6.0mm	3.0mm	IEHA663	EHA663		
	6.0mm	4.0mm	IEHA664	EHA664		
	6.0mm	6.0mm	IEHA666	EHA666		
	6.0mm	8.0mm	IEHA668	EHA668		
	7.5mm	3.0mm	IEHA673	EHA673		
	7.5mm	4.0mm	IEHA674	EHA674		

IEHA676

IEHA678

Global Headquarters 4555 Riverside Drive Palm Beach Gardens, FL 33410 1-800-342-5454 Outside The U.S.: +1-561-776-6700 Fax: +1-561-776-1272 www.biomet3i.com

SUBSIDIARIES

AUSTRALIA Phone: +61-2-9855-4444 Fax: +61-2-9888-9900

AUSTRIA Phone: +43-(0)6235-200-45 Fax: +43-(0)6235-200-45-9

BELGIUM Phone: +32-2-5410290 Fax: +32-2-5410291

BRAZIL Phone: +55-11-5081-4405 Fax: +55-11-5081-7484

CANADA Phone: +514-956-9843 Fax: +514-956-9844

FRANCE Phone: +33-1-41054343 Fax: +33-1-41054340

GERMANY Phone: +49-721-255177-10 Fax: +49-721-255177-73

IRELAND Phone: +353-1-800-552-752 Fax: +44-1628-820182

MEXICO Phone: +52-55-2282-0120 Fax: +52-55-2282-0120 ext. 20

THE NETHERLANDS Phone: +31-(0)78-629-2800 Fax: +31-(0)78-629-2801

NEW ZEALAND Phone: +64-508-122-221 Fax: +64-508-133-331

NORDIC REGION Phone: +46-40-17-6090 Fax: +46-40-17-6099

PORTUGAL Phone: +351-21-000-1647 Fax: +351-21-000-1675

ARGENTINA Phone: +34-93-470-59-50 Dentalmax, SA Fax: +34-93-372-11-25 Phone: +541-1482-71001

SWITZERLAND Phone: +41-(0)44-200-76-00 Fax: +41-(0)44-200-76-01

SPAIN

U.K.

TURKEY Phone: +90-212-573-6916 Fax: +90-212-662-7635 Phone: +44-800-652-1233 Fax: +44-1628-820182

Fax: +86-21-6329-1620 COLOMBIA 3i Colombia Phone: +571-612-9362

CHILE Cybel, SA

CHINA

Atek Inc.

Simply Go Online To www.biomet3i.com/signup

DISTRIBUTORS

Fax: +541-1482-67373

Phone: +56-2-2321883

Phone: +86-21-6329-1265

Fax: +56-2-2330176

Sign Up For BIOMET 3i's Electronic Newsletter "BIOMET 3innovations."

ARCHITECH PSR, Certain, Encode and Gold-Tite are registered trademarks of BIOMET 3/ LLC. BIOMET is a registered trademark and BIOMET 3/ and design are trademarks of BIOMET, Inc. © BIOMET 3/ LLC. All rights reserved.

Fax: +571-620-6412 COSTA BICA Implantec S.A. Phone: +506-234-9043 Fax: +506-224-7620

EL SALVADOR Dentimerc SA de CV Phone: +503-263-6350 Fax: +503-263-6676

GREECE Impladend Dental Implants, LLC Phone: +30-2310-501-651 Fax: +30-2310-522-417

7.5mm

7.5mm

6.0mm

8.0mm

ISRAEL H.A. Systems Phone: +972-3-6138777 Fax: +972-3-6138778

HONG KONG Ositek Inc., Ltd. Phone: +852-8121-6601 Fax: +852-3747-3754

ITAL Y Biomax, srl. Phone: +39-0444-913410 Fax: +39-0444-913695

JAPAN Implant Innovations Japan Phone: +81-66-868-3012 Fax: +81-66-868-2444

KORFA Jungsan *3i* Corp. Phone: +82-2-516-1808 Fax: +82-2-514-9434

LEBANON 3/ MENA s.a.I. Middle East And North Africa Phone: +961-1-694000 Fax: +961-1-694222

PARAGUAY Andres H. Arce y Cia SRL Phone: +595-21-208185 Fax: +595-21-496291

POLAND Dental Depot Phone: +48-71-341-3091 Fax: +48-71-343-6560

RUSSIA Com-Dental Phone: +7-495-797-6686 Fax: +7-499-242-9567

SINGAPORE Asia Implant Support & Services Phone: +65-6223-2229 Fax: +65-6220-3538



EHA676

EHA678

Rev A 08/08

SOUTH AFRICA Selective Surgical CC Phone: +27-11-991-7007 Fax: +27-11-672-1391

TAIWAN Kuo Hwa Dental Suppliers Co., Ltd. Phone: +886-2-2226-1770 Fax: +886-2-2226-8747

THAILAND 3i (Thailand) Co., LTD. Phone: +662-252-6685 Fax: +662-252-6686

UKBAINE Com-Dental Phone: +38-067-7007667 Fax: +38-044-5017117

UBUGUAY Pro3implant S.R.L. Phone: +598-2-4034163 Fax: +598-2-4034163