

- For Placement Of 5.0 And 6.0mm Implants In Medium Density Bone
- Alternating Lines And Bands Enhance Depth Mark Visibility And Drill Orientation In The Osteotomy
- Matte Finish Reduces Glare


Depth Mark Identification

The single line depth mark should be read at the center of the laser line/groove.

The broad band should be read at the beginning and end of the band/groove.

Recommended speeds:

- 1. Drill diameter 3.85mm = 1200 - 1500rpm
- 2. Drill diameter 4.85mm = 900rpm

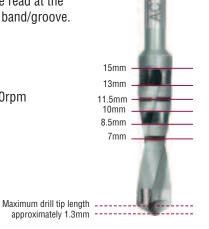

ACT Non-Irrigated Twist Drills

	Product Number	Diameter (mm)	Length (mm)	Tip Length (mm)
2mm	ACT3810	3.85	10.0	1.2
	ACT3815	3.85	15.0	1.2
	ACT3820	3.85	20.0	1.2
emm 6	ACT4885	4.85	8.5	1.3
	ACT4813	4.85	13.0	1.3
	ACT4818	4.85	18.0	1.3

Please see chart for drill tip length. Drill tip length should be considered in treatment planning as it is not included in depth mark dimensions.

Contact Your Local BIOMET 3i Representative To Place An Order For ACT 3.85mm And 4.85mm Drills Today!



Global Headquarters 4555 Riverside Drive Palm Beach Gardens, FL 33410 1-800-342-5454 Outside The U.S.: +1-561-776-6700 Fax: +1-561-776-1272 www.biomet3i.com

Sian Up For BIOMET 3i's Electronic Newsletter "BIOMET 3innovations." Simply Go Online To www.biomet3i.com/signup



ACT and Advanced Cutting Technologies are registered trademarks of BIOMET 3i, LLC. BIOMET is a registered trademark and BIOMET 3i and design are trademarks of BIOMET, Inc. ©2008 BIOMET 3i, LLC. All rights reserved.

REV A 04/08

SUBSIDIARIES

AUSTRALIA

Phone: +61-2-9855-4444 Fax: +61-2-9888-9900

Phone: +43-(0)6235-200-45 Fax: +43-(0)6235-200-45-9

BELGIUM

Phone: +32-2-5410290 Fax: +32-2-5410291

BRAZIL

Phone: +55-11-5081-4405 Fax: +55-11-5081-7484

CANADA Phone: +514-956-9843

Fax: +514-956-9844 FRANCE

Phone: +33-1-41054343 Fax: +33-1-41054340

Phone: +49-721-255177-10 Fax: +49-721-255177-73

IRELAND

Phone: +353-1-800-552-752 Fax: +44-1628-820182

Phone: +52-55-5679-1619 Fax: +52-55-5684-8098

THE NETHERLANDS Phone: +31-(0)78-629-2800 Fax: +31-(0)78-629-2801

NEW ZEALAND Phone: +64-508-122-221

Fax: +64-508-133-331

NORDIC REGION Phone: +46-40-17-6090 Fax: +46-40-17-6099

PORTUGAL

Phone: +351-21-000-1647 Fax: +351-21-000-1675

SPAIN

Phone: +34-93-470-59-50 Fax: +34-93-372-11-25

SWITZERLAND

Phone: +41-44-380-46-46 Fax: +41-44-383-46 55

TURKEY

Phone: +90-212-573-6916 Fax: +90-212-662-7635

Phone: +44-800-652-1233 Fax: +44-1628-820182

DISTRIBUTORS

ARGENTINA

Dentalmax, SA Phone: +541-1482-71001 Fax: +541-1482-67373

CHILE

Cybel, SA Phone: +56-2-2321883 Fax: +56-2-2330176

CHINA

Atek Inc. Phone: +86-21-6329-1265 Fax: +86-21-6329-1620

COLOMBIA

3i Colombia Phone: +571-612-9362 Fax: +571-620-6412 COSTA RICA

Implantec S.A. Phone: +506-234-9043 Fax: +506-224-7620

EL SALVADOR Dentimerc SA de CV Phone: +503-263-6350 Fax: +503-263-6676

GREECE

Impladend Dental Implants, LLC Phone: +30-2310-501-651 Fax: +30-2310-522-417

ISRAEL

H.A. Systems Phone: +972-3-6138777 Fax: +972-3-6138778

HONG KONG

Ositek Inc., Ltd. Phone: +852-8121-6601 Fax: +852-3747-3754

ITALY

Biomax, srl. Phone: +39-0444-913410 Fax: +39-0444-913695

Implant Innovations Japan Phone: +81-66-868-3012 Fax: +81-66-868-2444

KOREA

Jungsan *3i* Corp. Phone: +82-2-516-1808 Fax: +82-2-514-9434

LEBANON

3i MENA s.a.I. Middle East And North Africa Phone: +961-1-694000 Fax: +961-1-694222

PARAGUAY

Andres H. Arce y Cia SRL Phone: +595-21-208185 Fax: +595-21-496291

POLAND

Dental Depot Phone: +48-71-341-3091 Fax: +48-71-343-6560

RUSSIA

Com-Dental Phone: +7-495-797-6686 Fax: +7-499-242-9567

SINGAPORE

Asia Implant Support & Services Phone: +65-6223-2229 Fax: +65-6220-3538

SOUTH AFRICA

Selective Surgical CC Phone: +27-11-991-7007 Fax: +27-11-672-1391

TAIWAN

Kuo Hwa Dental Suppliers Co., Ltd. Phone: +886-2-2226-1770 Fax: +886-2-2226-8747

THAILAND

3i (Thailand) Co., LTD. Phone: +662-252-6685 Fax: +662-252-6686

UKRAINE

Com-Dental Phone: +38-067-7007667 Fax: +38-044-5017117

URUGUAY

Pro3implant S.R.L. Phone: +598-2-4034163 Fax: +598-2-4034163